

fPL

Feline pancreas-specific lipase

Vcheck



SPECIAL FEATURES

- Direct measurement of fPL concentration
- Now, start to measure cat's pancreatic lipase at your hospital!

REFERENCE RANGE (Measurement Range 1~50ng/ml)

NORMAL ≤ 3.6 ng/ml

EQUIVOCAL 3.6~5.4 ng/ml


ABNORMAL > 5.4 ng/ml



SAMPLE VOLUME

3 STEP PROCEDURE

1 Get 100µl of serum by using the capillary spuit

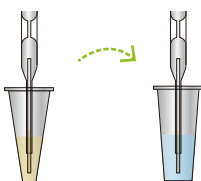

2 Using the tablet capillary spuit, mix well



3 Drop the mixed sample onto the kit

